


FLUORESCIN SODIUM OPHTHALMIC STRIPS USP



Worlds Largest Selling Fluorescein Sodium strips

FLUO STRIPS

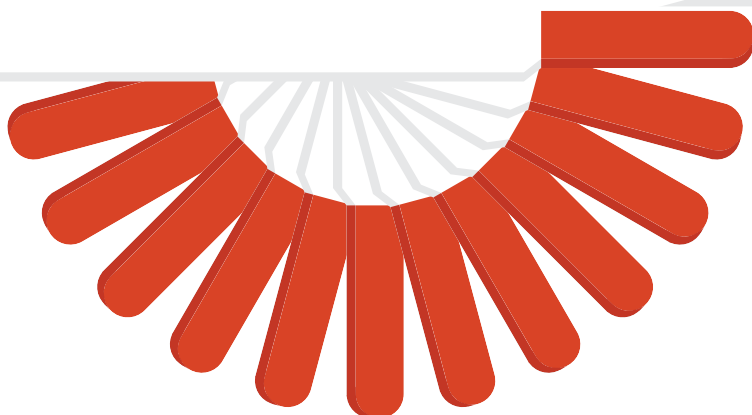

Overview - Fluorescein Sodium Strip is made of filter paper impregnated with highly pure fluorescein sodium solution. Sterilized by EO.

Use - It is the most commonly used diagnostic strip used to visualize defects or abrasions in the corneal epithelium by staining the areas of cellular loss. Also useful for evaluating hard contact lens fitting & applanation tonometry.

Direction - Moisten the tip of strip (the side with fluorescein sodium) by 1 or 2 drops of sterile water or sterile ophthalmic irrigating solution; touch the fornix or conjunctiva gently by the wet tip. For optimal results, the patient should blink several times after application.

Packing - Each strip includes 1 mg fluorescein sodium. 1 strip / pouch, 100 pouches/300 pouches per box.